

Dried Blood Spot Collection Instructions

Before proceeding to collection

Scan this QR code to see the Preparation and Collection instructions on your device



Sample Acceptance Criteria

If you are provided with multiple cards, please collect blood on all the provided cards.

✓ Examples of good DBS samples	
	Accepted
	Accepted
✗ Examples of bad DBS samples	
	No Second drop of blood to the same circle
	Fill the circle completely
	Fill all the circles

Vibrant Wellness
 www.vibrant-wellness.com | support@vibrant-wellness.com
 1-866-364-0963 (7 AM - 6 PM, Pacific Time)

Important Collection Tips!


Do's

- ✓ Drink a glass of water 30 minutes before collecting sample.
- ✓ Wash your hands in warm water for at least 1 minute right before the finger-prick. A warm shower before collection can also help. **This is very important to stimulate blood flow to the hands.*
- ✓ Let your hands hang at your side for 15-20 seconds.
- ✓ Shake your hands back and forth several times.

Don'ts

- ✗ Squeeze your fingertip as this will stop the blood flow.
- ✗ Touch your finger or rub or smear to the card.
- ✗ Apply more than one spot of blood per spot (no overlapping).
- ✗ Use heat to assist in drying.

Your kit includes:

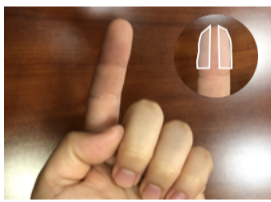

STEP 1

Choose the ring or pinkie finger on your non-dominant hand and clean the desired puncture site using the alcohol prep pad. Puncture site should be on the left or right side of the chosen fingers.



STEP 2

Unfold the blood spot card open on a flat surface. Remove and discard the plastic safety end on the blood lancet by twisting and pulling it.



STEP 3

Please note that the blood lancet will only activate and puncture once. With your palm facing up, press the blood lancet to your finger until you hear a click and feel a slight pinch.




STEP 4

Wipe the first drop of blood away with the gauze pad.



STEP 5


Press your finger down and forward from the base to increase blood flow.

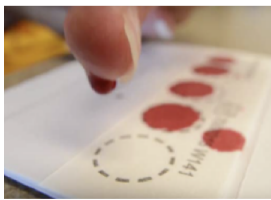

STEP 6

Allow blood to drip down onto the circles on the blood spot collection card. **Only One drop of blood per spot.** Ensure all five circles on the collection card are filled. Complete the entire collection process in a single day.

If you are provided with multiple cards, please collect blood on all the provided cards by repeating Step 5 and Step 6. You may use the additional lancet provided if needed.

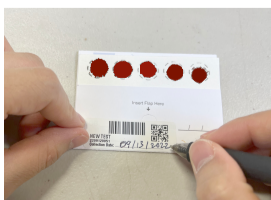

STEP 7

Let blood spot collection card sit out for 30 minutes or until completely dry. Apply band aid to finger when finished. **Write the date of collection on the blood spot collection card(s).**

NOTE

Place the card(s) inside the biohazard bag and place the biohazard bag inside the return envelope. Mail immediately via FedEx. Holding collected samples more than **24** hours may result in rejection.

