

Your kit includes:



Urine collection cup
x1



Specimen tube
x1



Pipette
x1



Collection label
x1







Biohazard bag
x1



Return Envelope
x1


Sample Acceptance Criteria

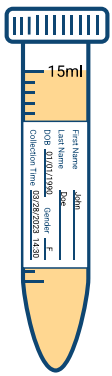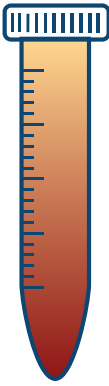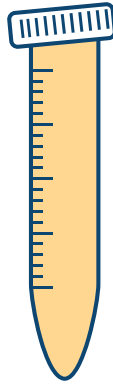✔ Accepted tube&label	✘ Blooded and/or overfilled	✘ Improperly sealed
		

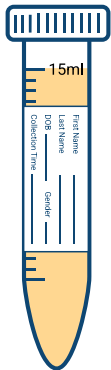
✘ Insufficient labeling(<i>missing label, date and time of collection</i>)
 <div data-bbox="779 1669 1101 1816" style="border: 1px solid black; padding: 5px; margin-left: 200px;"> <p>Blank</p> <p>First Name _____</p> <p>Last Name _____</p> <p>DOB _____ Gender _____</p> <p>Collection Time _____</p> </div>




Step 1
Collect your urine in the urine collection cup provided until the fill line.

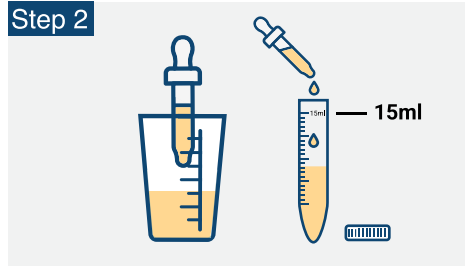

Step 2
Unscrew the cap from the specimen tube. Use the pipette to transfer the urine from the cup to the tube until it reaches the 15 ml line.

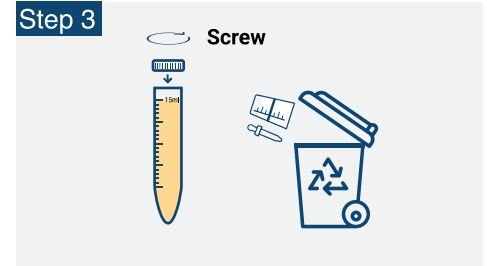


Step 3
Screw the required specimen tube securely by twisting tightly. Please discard the cup and pipette after use.

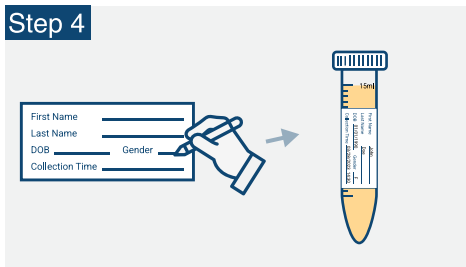


Step 4
Write your Name, Date of Birth and Collection Time & Date on the provided label and stick it on the specimen tube.

***This Step is important as any missing information will delay the processing of your specimen.**

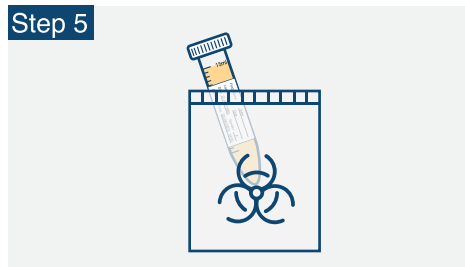

Step 5
Place the specimen tube required into the biohazard bag with the absorbent pad and seal completely.

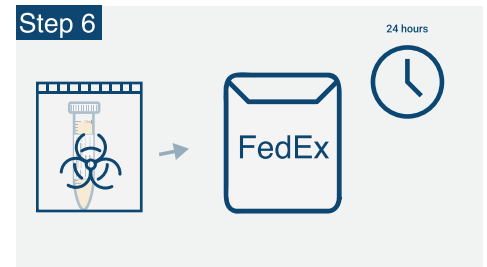

Step 6
Place the biohazard bag inside the return envelope

Mail immediately via FedEx. Holding collected samples more than 24 hours may result in rejection.